Essix[®] Bleach Tray & Model Duplication Material

DIRECTIONS FOR USE
INSTRUCCIONES DE USO
MODE D'EMPLOI
ISTRUZIONI PER L'USO
GEBRAUCHSANWEISUNG

Manufacturer:
DENTSPLY International Raintree Essix
7290 26th Court East
Sarasota, FL 34243 USA
Tel: 800.883.8733
www.essix.com

Essix® Bleach Tray & Model Duplication Material

(Instructions for use on Essix® Vacuum Machines only*)

ENG

Intended Use:

Essix® Bleach Tray & Model Duplication Material is used to fabricate intra-oral tooth whitening (bleaching) trays or to fabricate a mold of an existing stone model.

Indications for Use:

Essix® Bleach Tray & Model Duplication Material is indicated for a patient undergoing a tooth whitening procedure or when a duplicate of an existing stone model is required.

Contraindications: This product is contraindicated for patients with a history of allergic reactions to plastics.

⚠ Warnings:

Excessive heating time during thermoforming will cause plastic to over sag and potential for injury to operator.

Improper thermoforming of the plastic appliance may cause the appliance to crack/break, resulting in sharp edges, loose pieces and possible aspiration of pieces.

Precautions:

Patients with a history of allergic reactions to plastics should not use this product.

Store in a cool, dry place.

Single Use Only. Risks of reuse include infection and cross-contamination to patient and healthcare provider as well as other wear-related irregularities.

Adverse Reactions: Allergic reactions.

Step by Step Instructions:

1. All models should be trimmed to $\frac{3}{4}$ " high. Full arch models should have the palate trimmed away.
2. IMPORTANT: Pre-heat the machine before using! Pre-heat the vacuum machine for at least 3 minutes or until the heating element is a solid orange color. (Heating times may vary with current fluctuations or length of time the heating element has been left on.)
3. Spray the model with Trim-Rite® Release Agent to prevent the material from sticking to the model.
4. Place the plastic into the frame of the vacuum machine and the model on the base plate.
5. Swing the heating element over the plastic. Observe the material as it heats.
6. Material will sag approximately $\frac{1}{2}$ " - this is the optimal thermoforming time.
7. Move the heating element away from the plastic.
8. Turn on the vacuum and lower the plastic over the model.
9. Immediately spray the plastic encased model with Essix Freeze Spray® Coolant. Not only does it cool the plastic, prohibiting further thinning, but it "shrinks" the material around the cast, realizing a tighter fit.
10. Let vacuum continue to run until plastic is cool.
11. Use fabricated mold to pour die stone into.

*Heating time may vary depending upon machine. If plastic does not adapt to model, add heating time until adaptation occurs. If plastic "folds" when thermoformed, reduce heating time until material does not fold.

Thickness	Vacuum Heat Time**	Biostar™ Code**	Essix® SelectVac®	Drufomat
1.0mm (.040")	30 seconds	142	1:30	1:10
1.5mm (.060")	40 seconds	162	1:35	1:15
2.0mm (.080")	45 seconds	200	1:40	1:25

**after preheating element

Biostar and Drufomat are not trademarks of Dentsply Sirona.

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