Retainer Brite® Foam Mousse Retainer Brite®

DIRECTIONS FOR USE MODE D'EMPLOI

Description: Retainer Brite® Foam is a patented formulation designed to whiten teeth and freshen orthodontic aligners and clear retainers (appliances). Retainer Brite® Foam utilizes Anionic Technology powered by 3.8% hydrogen peroxide to help eliminate 99.9% of odor-causing bacteria on removable dental appliances* when used outside of the mouth.

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Indications for Use: Indicated for users of aligners and retainers who need to clean their appliances (outside of the mouth). The product is also indicated for tooth whitening (inside of the mouth).

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⚠ Warnings:

- If irritation (such as redness, swelling, soreness) of the gums or mouth occurs, discontinue use and consult a dentist.
- Adult supervision is required for use by children under 12.
- Avoid contact with eyes, skin or hair; flush with water if contact is made.

- Avoid swallowing; do not induce vomiting.
- Do not apply to gums or under tongue.
- Do not use if nursing or pregnant.
- Use only with orthodontic clear aligners, clear retainers and clear/hard mouth pieces.
- Prolonged use may require dentist supervision.

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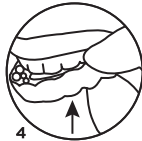
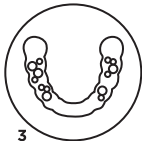
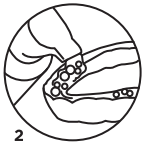
Centres antipoison (Europe) :

echa.europa.eu

Adverse Reactions: Allergic reaction.

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**Directions - Tooth Whitening
(Inside of the Mouth):**

1. Remove appliance.
2. Dispense one-half to one full pump of foam inside each appliance.
3. Spread foam evenly using a finger.
4. Place appliance back into your mouth and spit out excess foam.
5. Wipe off fingers. Use up to four times daily to whiten teeth and freshen appliances.

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Directions - Appliance Cleaning (Outside of the Mouth):

1. Remove appliance
2. Place appliance inside storage case.
3. Cover appliance with foam and let sit for up to 5 minutes.
4. Rinsing appliances before replacing them in the mouth is optional.

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Storage Conditions:

- Keep away from sunlight and heat as contents may pressurize.

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Dentsply Sirona Orthodontics Inc.
Eversmile, Inc.